



Excel Formula®

Baseplate Wax

Directions for Use

Model Forming: Take a sheet of wax and tear in half, soften the sheet of wax over a Bunsen flame and press it onto the plaster model. Make sure the wax goes into all the curves and cracks of the model and then trim/carve the excess wax away using a heated knife.

Other waxes: Take a sheet of wax and soften over a Bunsen flame, and mold the wax into a block for curved Bite Rim, also can be suitable for bite registration.

Tips: Baseplate wax texture allows for sharp and clean carving at room temperature using normal dental carvers and normal dental laboratory technique.

Trimming/Carving: The wax should trim easily and cleanly without tearing or chipping.

Storage: Keep packaged in a dry place protected from the sun and far from any heat source.

Safety: For full disclosure of Safety Data Sheets (SDS), please visit our website at **www.StGeorgeTechnology.com**, download SDS or SDS directly with our QR code, or call **(910) 397-0781** to arrange a hard copy to be sent to you.

View SDS Here:



St. George Technology
Performance Polymers



facebook.com/StGeorgeTechnology

@saintgtech

PO Box 12410 • Wilmington, NC 28405 • USA
p) 910.397.0781 • f) 910.397.0791
www.StGeorgeTechnology.com