



Excel Formula®

Bite Rims

Directions for Use

Application: To use in denture fabrication process.

Properties: Very good shape, not too bulky. Small recessed cavity on bottom for easy adaptation to Baseplate/model. Nice firm stability for better setup and accuracy.

Directions for Use: Place Bite Rim in warm water for around 1 minute. This will allow it to become malleable and easily adapted to the Baseplate/model. Bite Rims can also be softened by flame from a burner.

Storage: Keep packaged in a dry place protected from the sun and far from any heat source.

Safety: For full disclosure of Safety Data Sheets (SDS), please visit our website at www.StGeorgeTechnology.com, download our SDS directly with our QR code, or call (910) 397-0781 to arrange a hard copy to be sent to you.

View SDS Here:



St. George Technology
Performance Polymers



facebook.com/StGeorgeTechnology

@saintgtech

PO Box 12410 • Wilmington, NC 28405 • USA
p) 910.397.0781 • f) 910.397.0791
www.StGeorgeTechnology.com